

4-Hole Adapter Spacer Plate Instructions

WHAT'S IN THE BOX

One of the following:

- 4-Hole Adapter 1/8" Spacer Plate (700-300) with 4 M6 x 20 mm Dacromet Flat Head Capscrews
- 4-Hole Adapter 3/8" Spacer Plate (700-375) with 4 M6 x 25 mm Flat Head Capscrews

Instructions

INSTALLATION

Attach the Spacer Plate to the desired 4-hole component using screws of the appropriate length and strength. Apply Loctite 242 (or equivalent) and tighten the screws to 9 ft-lbs (12 Nm).

Note: if desired, multiple Spacer Plates may be installed, with the following constraints:

1. The total height of the Spacer Plates must not exceed 3/4" (1.9 cm).
2. The Spacer Plates plus any components connected to them (for example, an Alpha Lock or Alpha Locking Lanyard) must be held securely by screws that do not exceed 50 mm in length.

WARRANTY

The warranty for the 4-Hole Adapter Spacer Plate is one year from the date of invoice. A product returned during the warranty period will be evaluated and repaired or replaced at WillowWood's discretion. Use of a 4-Hole Adapter Spacer Plate for amputees whose adjusted body weight is more than 250 pounds (115 kg) or who engage in extremely high and abusive activities is against WillowWood's recommendation and will void the one-year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads normally or routinely carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in an injury to an individual's natural foot.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

WillowWood®

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands

