

LimbLogic® Communicator

Version 2015 Q2 Release Notes

LimbLogic Communicator version 2015 Q2 (2015.2.x.x) is a focused update released in **Spring 2015** which includes additional features and fixes for reported issues.

For information on changes made in previous versions, please see the [LimbLogic Release Notes Archive](#).

For detailed information on using any of LimbLogic Communicator's features, click the Guides tab in the Communicator software to view the LimbLogic Communicator User's Guide.

Additional LimbLogic product information can be found on the [LimbLogic product page](#).

1. REQUIREMENTS

LimbLogic Communicator version 2015 Q2 requires Windows 7 or later. Support for Windows XP has been discontinued.

2. ADDITIONS

Adaptive Mode: LimbLogic switches between a higher and a lower vacuum level range, depending on whether the patient is moving or resting. Because the pump activates less frequently at the lower vacuum level, the patient may find Adaptive Mode helpful in quiet environments or during periods of prolonged sitting. Using this version of LimbLogic Communicator software, Adaptive Mode can be setup for the patient's use.

Assistant: the Assistant function allows you to view information about socket conditions that have been found to correlate with socket motion. This function collects and processes gait data to detect several of these features and then combine them to create a single indication of motion. This function works well with patients with strong, symmetrical gait. Use with patients with highly compensatory gait will have less accurate results. A qualified professional should always confirm indications from the Assistant.

3. REMOVALS

No major features have been removed.

4. CHANGES

No major changes have been made to existing functionality.

5. FIXES

A variety of fixes have been made to address LimbLogic system discovery and registration, support for backwards compatibility with existing LimbLogic systems, and general reliability and stability of software.

WillowWood

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com



Ohio Willow Wood Company B.V.
Keizersgracht 62/64
1015 CS Amsterdam
The Netherlands
Patent www.willowwoodco.com/education-and-resources/patents

