Device Description

Designed for comfort and performance, the Alpha Truseal Liner is made from proprietary, platinum cured grade silicone. The four integrated fins are designed to ensure confident suction suspension while the satin finish allows easy donning and doffing

WHAT'S IN THE BOX

Alpha Truseal	S494-11XX
	S495-11XX

**Value of X depends on size ordered.*

WILLOWWOOD®

WillowWood Global LLC
15441 Scioto Darby Road
Mt. Sterling, OH 43143

téléphone 740.869.3377 / 800.848.4930

télécopieur 740.869.4374 www.willowwood.com

Brevet www.willowwoodco.com/education-and-resources/patents

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Electronic Instructions for Use are available at www.willowwood.com.
A paper copy of the Instructions for Use is available upon request at no additional cost.

DONNING

- Invert the liner such that the fins are on the inside
- Place the liner against the bottom of the limb
- Roll the liner onto the limb taking care to avoid trapping air or creating wrinkles
- Step into your socket and allow air to be expelled from the expulsion valve

MAINTENANCE AND CLEANING

- Inspect daily for holes, tears, and premature wear
- Keep the surfaces clean and clear of debris using mild soap and dry with a lint free cloth
- Disinfect the surfaces weekly using isopropyl alcohol and a lint free cloth

WARRANTY

WillowWood Liners are warrantied for 12 months, against defects in material and workmanship, from the date of patient fitting.

REPORTING A SERIOUS INCIDENT

In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient's, user's or other person's state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/user is established.

Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.

This Alpha Product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.

WARNING: To avoid danger of suffocation, keep this product away from babies or children.

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