# WillowWood<sup>®</sup>

## ALPHA TrūSeal<sup>™</sup>

### Patient Instructions

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Thank you for using this liner. In order to ensure the highest level of comfort and longest possible life from your liner, please take the following steps:

- Read and follow these instructions.
- Inspect your liners after each day of use. Watch for signs of unusual wear. This may indicate changes in the prosthetic fit. If you see unusual wear, call your prosthetist.
- Call your prosthetist if you have any questions about your liners.

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Failure to follow WillowWood instructions will void the warranty.

Since WillowWood believes that the liners must be fit and dispensed by a qualified professional, please understand that WillowWood will not ship liners directly to an amputee's home.

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*If you do not understand these warnings, please discuss them with your prosthetist.*

-  Caution: Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your liner product to these conditions. Doing so may result in ignition resulting in injury or death.
-  This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.
-  WARNING: To avoid danger of suffocation, keep this product and its packaging away from babies or children, and do not put the product on your head.
-  Do not expose the liner to items to which the patient is allergic. The liner could absorb the allergen and become a permanent source of exposure.
-  Silicone swells in contact with Thermoplastic Elastomer (TPE). Do not allow products made of TPE to contact the liner.
-  Consistent contact of the sealing rings against the socket wall is needed to maintain prosthetic suspension. If any dirt, lint, water, or other material disrupts the seal from contacting the socket wall, prosthetic suspension could become compromised.

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## DONNING

**Do not apply any type of lotion or powder to the residual limb or to the liner, as these products could damage the liner. The residual limb should be clean, dry, and free of soap residue. Cover any open wounds or non-intact skin with a bandage or some other appropriate covering to prevent direct contact between the wound and the liner.**

1. Invert the liner so that the fins are on the inside.
2. Place the end of the liner against the end of the limb. The orientation of the logo does not matter, but varying the orientation can prolong the life of the liner (unless cut to fit trim lines).
3. Carefully roll the liner up onto the limb. To prevent irritation of the limb, make sure that there are no wrinkles or air pockets between the limb and the liner.

**DO NOT PULL the liner onto the limb. Pulling the liner may stretch the skin and result in discomfort or skin irritation.**

**WARNING: If fabric socks or other wicking materials are used on the outside of the liner, do not cover the sealing fins. Covering the sealing fins will create a leak path and could cause loss of suspension.**

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## CLEANING AND CARE

Proper hygiene is extremely important when using the liner. The liner must be thoroughly cleaned at the end of each day of wear and disinfected once each week.

### Daily Cleaning

Thoroughly clean the side of the liner without the fins. Use lukewarm tap water and a mild body soap that does not irritate the skin. Apply the soap with a clean, soft cloth or sponge.

Rinse **all** the soap residue from the liner with water.

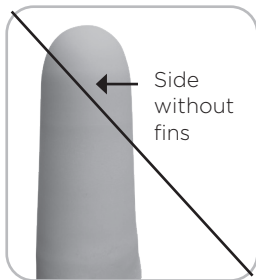
Dry the liner with a clean, lint-free cloth.

Place the liner, **with the fins on the outside**, onto the Drying Stand\* or onto a clean, dry, flat surface.

Clean the residual limb with soap and rinse well.

*\*Available separately in most countries*

**Caution: Drying the liner with the fins facing in will damage the liner.**

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## Weekly Disinfection

Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth.

Lightly wipe the side of the liner without the fins with the cloth for two minutes.

Rinse off any excess alcohol with water. Dry the liner with a clean, lint-free cloth.

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**Notes: Do not immerse the liner in ethyl or isopropyl alcohol. Extended contact with large amounts of ethyl or isopropyl alcohol will stiffen the liner.**

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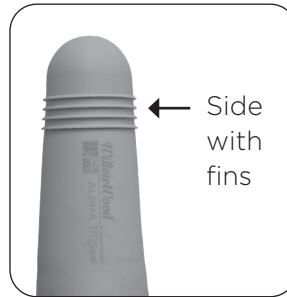
## Daily Inspection of the Liner

Inspect the liner after each day of use.

- Watch for signs of unusual wear. This may indicate changes in the prosthetic fit.
- Check for torn fins.
- Look for debris in the fins.

If you see any of these conditions, call your prosthetist as soon as possible.

## Perspiration

Some activities could cause you to sweat heavily. If this happens, occasionally dry off the liner and the limb. This helps to keep the liner from slipping on the limb.

## Storage

When the liner is not being worn, it should be stored with the fins on the outside in a cool, dry place on the Drying Stand or in the liner's original box. If the liner will not be used for an extended period, cover the dry liner with a clean plastic bag.

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## TROUBLESHOOTING

<b>Problem</b>	<b>Probable Cause</b>	<b>Solution</b>
Residual limb and liner rotate inside the socket.	Residual limb is extremely cylindrical, and socket is loose.	Tighten socket fit.
Top edge of liner is rolling down the leg.	Liner is possibly too short or too tight.	Call your prosthetist.
Open wounds and non-intact skin.	Call your prosthetist right away.	Follow the instructions your prosthetist gave you. Never use a liner in direct contact with open sores or non-intact skin. If you must wear the liner over open sores or non-intact skin, wear a bandage over the wound.
Heavy sweating after wearing the liner a long time without sweating.	Air is getting between the liner and the residual limb. This is possibly because your limb has shrunk.	Call your prosthetist.
	There is a hole in the liner.	Call your prosthetist.
Heavy sweating (if you have just recently started wearing the liner).	Your body has not yet adjusted to the liner.	If you are still sweating after several weeks of liner use, call your prosthetist.
Skin irritation all over your limb.	Cleaning or disinfecting your liner the wrong way.	Make sure that you are following the proper cleaning, rinsing, and disinfection procedures outlined in this pamphlet.
	Using a harsh or irritating lotion, cream, powder, or soap.	Have you recently started using a different lotion, cream, powder, or soap? If so, stop using that product.
	Washing the limb too frequently.	Washing the limb more than once per day can result in increased levels of skin irritation.
	Infection of the limb.	Call your prosthetist. You may also need to call a dermatologist.
	The liner or the socket is loose, possibly because your limb has shrunk.	Call your prosthetist
Skin irritation along the top edge of your liner.	The liner is being pulled onto the limb instead of being rolled onto the limb, or the liner is too tight.	Refer to the instructions in this pamphlet for putting on your liner. If the problem continues, call your prosthetist.

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Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands/Países Bajos  
Patent/Patente [www.willowwood.com/education-and-resources/patents](http://www.willowwood.com/education-and-resources/patents)

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WillowWood Global LLC