

LimbLogic® Quick-Start Guide

1. Turn on the Vacuum Pump.



Press one time.
Pump will beep once.



Wait 10 seconds.


2. Turn on the fob.



Press and hold until digits flash in "chasing" pattern. Fob displays current vacuum level. Pump is Active and will operate until preselected vacuum level has been reached.

3. To enter Standby Mode:



Press for approximately 2 seconds. Chasing pattern will be displayed, then Standby Indicator will light. 

To resume vacuum regulation:



Press for approximately 2 seconds. Chasing pattern will be displayed as pump returns to Active State.

4. Turn off the Vacuum Pump.



Press one time.
Pump will beep either two or four times.



Fob Guide


Vacuum Pump Battery Indicator

 50-100%

 30-50%

 20-30%

 < 20%

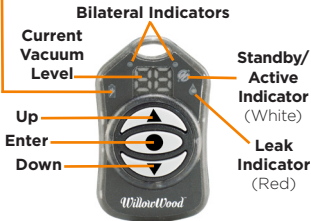

Refer to full manual for complete instructions.

WillowWood

The Ohio Willow Wood Company
15441 Scioto Darby Road
Mt. Sterling, OH 43143
phone 740.869.3377 / 800.848.4930
fax 740.869.4374 www.willowwoodco.com

Ohio Willow Wood Company B.V
Keizersgracht 62/64
1015 CS Amsterdam, The Netherlands
Patent www.willowwoodco.com/education-and-resources/patents

PN-2015-1

24 MAR 2015

