# Meta<sup>®</sup> Arc

## DEVICE DESCRIPTION

The META<sup>®</sup> Arc is a prosthetic foot that features a polycentric ankle and unibody composite construction. The polycentric ankle enhances stability and stabilizes socket forces. The META Arc is waterproof and designed for non-invasive, single patient use.


## INDICATIONS FOR USE

- Moderate to high activity lower extremity prosthesis users

## CONTRAINDICATIONS FOR USE

- Patients who do not have clearance for the foot.
- Patients who exceed weight rating of the foot.
- The Meta Arc is not intended for use during high impact sports.
- The Range of Motion provided by the ankle may be diminished if a continuous cosmetic cover is applied.

## ALIGNMENT

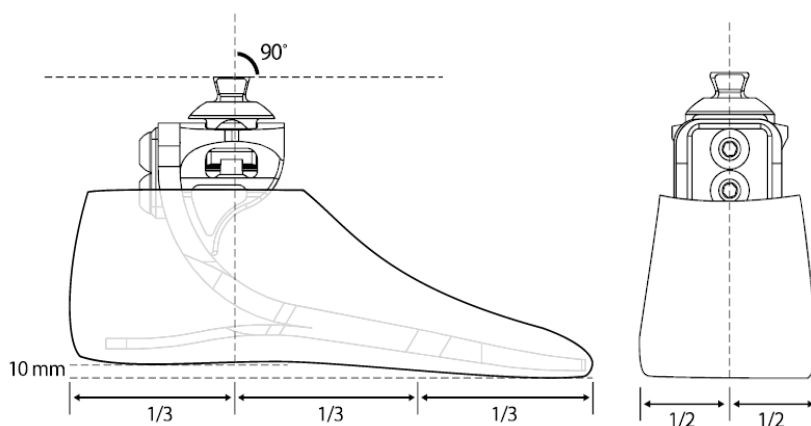**WillowWood<sup>®</sup>**

WillowWood Global LLC  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
téléphone 740.869.3377 / 800.848.4930  
télécopieur 740.869.4374 www.willowwood.com  
Brevet www.willowwoodco.com/education-and-resources/patents

Emergo Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

## **MAINTENANCE AND CLEANING**

If you expose the Foot to sand, dirt, debris, or salt water:

1. Remove the Foot Shell and Foot Sock.
2. Clean out the debris.
3. Clean the Foot, the Foot Shell, and the Foot Sock with body soap and water.
4. Allow the Foot Shell and Foot Sock to dry before reassembling.

Rinse the Meta Arc ankle unit with fresh water or compressed air to remove excess debris. It may help to use your hand to rotate the ankle unit while rinsing with fresh water or blowing air.

The foot is not field serviceable and there is no recommended maintenance schedule.

## **WARRANTY**

The warranty for the Foot is three years from date of patient fitting. Use of the Foot for amputees whose modified body weight is more than 166 kg (365 lbs) or who engage in extremely high and abusive activity is against WillowWood's recommendations and will void the three year warranty. Modified body weight is defined as the weight of the amputee plus any loads carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural feet; and activities that expose the prosthesis to corrosives such as salt water without adequate rinsing following exposure.

## **WARRANTY DISCLAIMER**

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

## **WILLOWWOOD RETENTION OF RIGHTS**

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

## **REPORTING A SERIOUS INCIDENT**

In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient's, user's or other person's state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/user is established.

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