FINNISH

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Warranty Disclaimer

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood’s liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WillowWood Retention of Rights

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

ENGLISH

REPORTING A SERIOUS INCIDENT

In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient’s, user’s or other person’s state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/ user is established.

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Emργο Europe  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands

Electronic Instructions for Use are available at [www.willowwood.com](http://www.willowwood.com). A paper copy of the Instructions for Use is available upon request at no additional cost.

WILLOWWOOD.

ALPHA®

Prosthetist Information

| WHAT’S IN THE BOX |                         |
|-------------------|-------------------------|
| Alpha Liner       | Prosthetist Information |
|                   | Patient Instructions    |

Before using the Alpha Liner, please read the Instruction Manual available at willowwood.com. Provide the amputee with the **Patient Instructions** included in the box.

IF USING AN ALPHA LOCKING LINER

- Provide a locking pin for each Alpha Locking Liner.
- Use a locking pin with M10-1.5 threads (or 1/4”-20 threads for pediatric liners).
- Apply LocTite Removable Threadlocker 242 (or equivalent) to the pin threads prior to installing the pin.

**Warning: Failure to ensure that the locking pin is secure in the Locking Liner could result in the pin coming out of the liner, resulting in the potential for injury to the amputee.**

Note: WillowWood recommends using an additional suspension method such as a suspension sleeve and one of the following lock options:

- Alpha Lamination Lock (700-AL600)
- G-Lock (700-GL480)
- Alpha Lock (700-AIS480)
- Mini G-Lock (700-MGL480)
- Alpha Locking Lanyard (700-AIS100)

 **Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.**

 **This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.**

 **Warning: To avoid danger of suffocation, keep this product away from babies or children.**

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