WillowWood[®]


ALPHA[®]


Prosthetist Quick Reference Guide	2
Notice Technique pour le Prothésiste	6
Kurzanleitung für Orthopädietechniker	10
Guía Breve de Referencia Prostética	14
Korte Handleiding Instrumentmaker	18
För Ortopedingenjören Snabb Referens guide	22
義士装具士向けリファレンスガイド	26
Prothésiste Guide de référence rapide	30

PRE-STRETCHING CLASSIC LINERS (IF NECESSARY)

Note: Silicone Liners are not heat-moldable. It should not be necessary to heat stretch Hybrid or Duo Liners.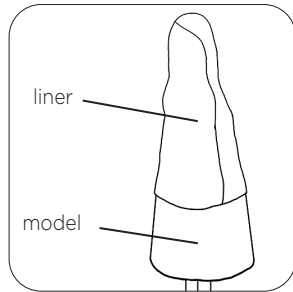

Roll the Alpha onto a positive model of the desired volume. Leave it there for one hour, OR...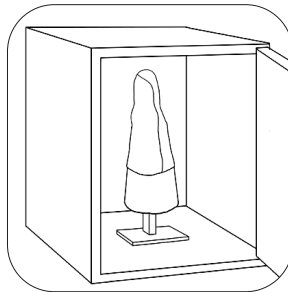

Place the model and liner into a 65°C oven for 2-5 minutes. If necessary, increase the amount of time in the oven by 5-minute increments, but do not exceed 15 minutes total. Be sure to use a positive model that can withstand the customizing temperature. Allow the liner to cool for 10 minutes.

Patients wearing Alpha Hybrid Liners with a progressive profile can be retrofit with Alpha Silicone Liners with a progressive profile. Patients wearing Alpha Hybrid Liners with an AK Symmetrical Profile can be retrofit with Alpha Silicone Liners with an AK Symmetrical Profile. Consult the Ordering Guidelines for more information.


DONNING


Invert the liner. Place the end of the liner against the end of the limb. Orient as follows:

Carefully roll the liner up onto the limb.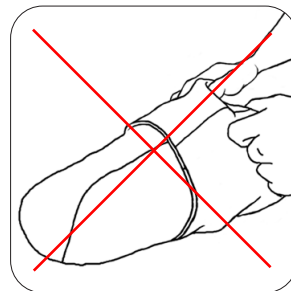

DO NOT PULL the liner onto the limb.

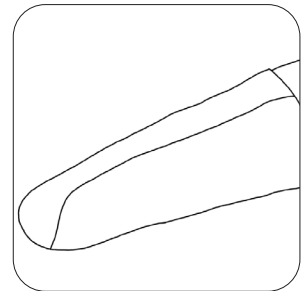Make sure that there are no wrinkles or air pockets between the limb and the liner.

Liner	Logo Placement
Alpha Classic® AK Liner Alpha AK DESIGN® Liner	Logo on lateral side (outside).
Alpha Hybrid® AK Liner Alpha Silicone® AK Liner Alpha SmartTemp® AK Liner WillowWood® Express AK Liner Alpha Duo® Liner Alpha Basic™ Liner	Logo position does not matter, but varying the orientation can prolong the life of the liner.
Other Alpha liners	Logo in front.

DONNING THE ONE GEL SOCK (FOR ALPHA DUO LINER)

Invert the One Gel Sock so that the silicone side is facing out. Place the end of the inverted sock against the end of the liner.

Roll the sock up onto the limb so that the silicone side is against the liner. Make sure that there are no wrinkles or air pockets between the One Gel Sock and the liner.

Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.

This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.

Warning: To avoid danger of suffocation, keep this product away from babies or children.

IF USING AN ALPHA LOCKING LINER

- Provide a locking pin for each Alpha Locking Liner.
- Use a locking pin with M10-1.5 threads (or 1/4"-20 threads for pediatric liners).
- Apply Loctite Removable Threadlocker 242 (or equivalent) to the pin threads prior to installing the pin.

Warning: Failure to ensure that the locking pin is secure in the Locking Liner could result in the pin coming out of the liner, resulting in the potential for injury to the amputee.

Note: WillowWood recommends using an additional suspension method such as a suspension sleeve and one of the following lock options:

- Alpha Lamination Lock (Part No. 700-AL600)
- Alpha Lock (Part No. 700-AIS480)
- Alpha Locking Lanyard (Part No. 700-AIS100)
- G-Lock (Part No. 700-GL480)
- Mini G-Lock (Part No. 700-MGL480)

CAPTURING THE SHAPE


If you use the **OMEGA**® System, refer to the **OMEGA**® User's Guide for directions.

If making a hand cast, follow these steps:


Don the Alpha Liner. Cover with plastic protection and a cast sock, then apply gypsum tape over the cast sock.

Press only on the two sides of the patella tendon.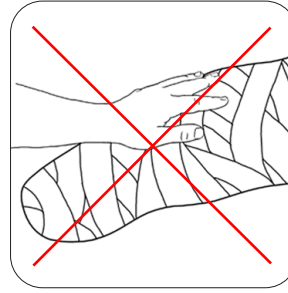

Do not influence the shape.

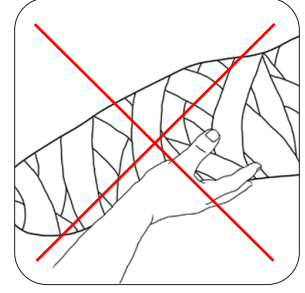Do not push in on the posterior wall at all.


CAST MODIFICATIONS (NO BUILD-UP ON THE POSITIVE - NEVER)

Reduce the positive model as follows to obtain Total Surface Weight Bearing:

Uniform, Tapered, Contoured			Progressive	AK
3 mm	6 mm	9 mm		
3-5%	5-7%	7-9%	5-7%	3-6%


To reduce a plaster positive:

Goal: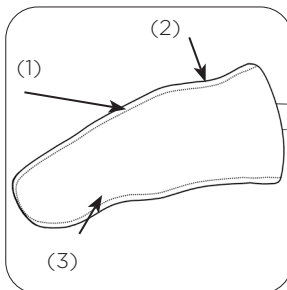


- (1) symmetric layer removed
- (2) original positive
- (3) modified positive for an Alpha socket


Method: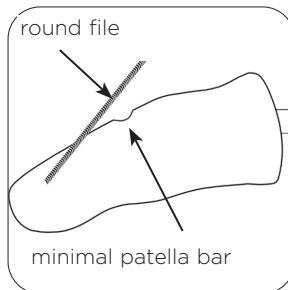

Carve a patella bar that is straight and no deeper than the depth of your finger marks. (Patella bar is optional.)

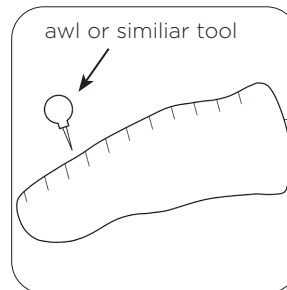
Make a series of reference marks along the model, approximately 3 cm apart. Pierce the model at each of the marks.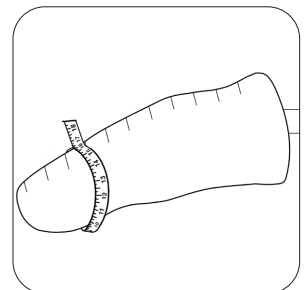


Measure and record the circumference at each of the marks. Reduce each circumference by the amount listed above to obtain the goal circumferences.


Note:
The suggested reduction amounts are starting points. You may need to reduce slightly more or less based on casting technique and limb conditions.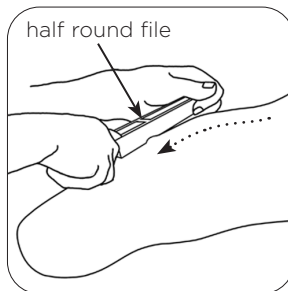

Using long continuous strokes, repeated 3 or 4 times at each spot, remove an even layer everywhere, following all contours without changing them. Do NOT reduce the length. Having reduced the whole surface, re-measure at each awl hole. Repeat until you have achieved the goal circumferences.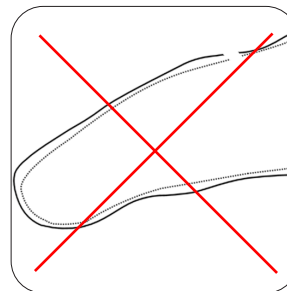

Wrong modification (plaster was not removed symmetrically).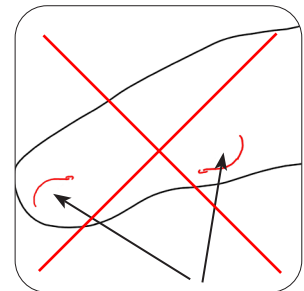

Do not add plaster - not even at the fibula head or at the distal tibia.


SOCKET EVALUATION


For AK sockets only: Attach Velcro®-type material to the anterior and lateral sides of the socket, 4 cm below the trim line.

For AK sockets only: Fold the edge of the liner over the top of the socket and attach to the Velcro-type material.

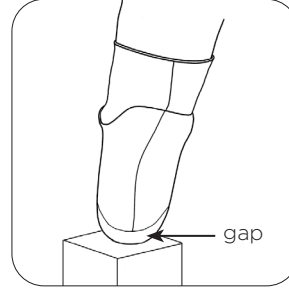
The socket is **TOO TIGHT** (or the socket is too long) if the limb does not contact the distal end of the socket in full weight bearing.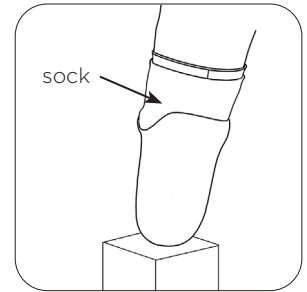


The socket is **TOO LOOSE** if the amputee can don a sock over the liner and slide down easily to the distal end of the socket.

Note: Velcro®-type material does not stick to the one-way stretch fabric of the AK Hybrid Liner.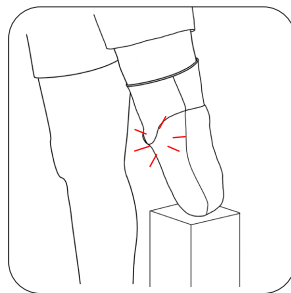

Check to see if the socket is tight around the edges of the patella when the leg is at full extension. If socket is tight around patella at full extension, ease out the edges.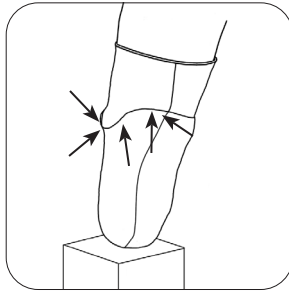

Ease out the edges of the socket everywhere so that the brim does not "saw" into the liner with the normal movements of the leg.


The socket **FITS CORRECTLY** if:

- 1) the limb begins to make slight distal contact under full weight bearing,
- 2) the limb contacts the distal end of the socket without socks on the liner
- 3) adding one single-ply sock over the liner prevents the limb from contacting the distal end of the socket under full weight bearing.


The inside of the socket must be perfectly smooth with no sharp edges. If a hole is drilled in the distal end of the socket, make sure that the edges have no sharp burrs that can snag the fabric, and create a flush surface by installing a Poron Filter (included with WillowWood's Suction Pyramid ad LimbLogic® 4-Hole Adapter, and also available separately as item number 700-AIS487).

TRIMMING


If desired, trim the Alpha to a shorter length (for a BK, no shorter than 8 cm above the medial/lateral brim; for an AK, not within 3 cm of where the Velcro-type material attaches). To help prevent the seam from unraveling, place a small drop of instant adhesive on the seam directly below the trimmed edge.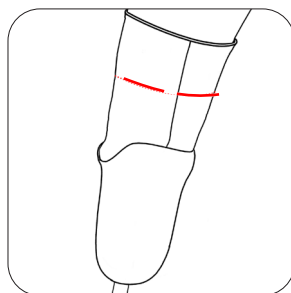


Trim straight across.

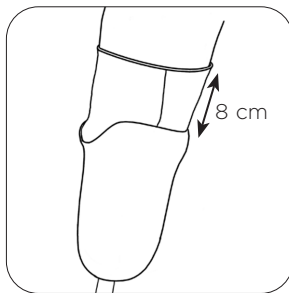GOOD

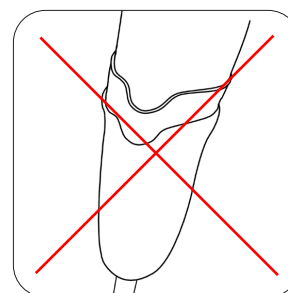DO NOT FOLLOW THE CONTOUR.

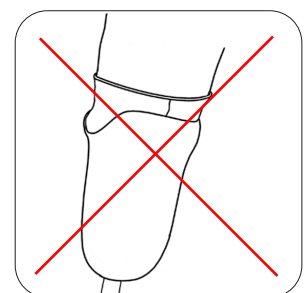TOO SHORT.

To help prevent the seam from unraveling, place a small drop of instant adhesive* on the seam directly below the trimmed edge.


To repair an unraveling seam, place a small drop of instant adhesive* on the seam directly below where the seam has begun to unravel.

*Before the instant adhesive completely soaks into the seam, apply a drop of accelerant directly on top of the adhesive. If you do not apply any accelerant to the instant adhesive, wait until the instant adhesive sets up before using the liner.

CLEANING (AFTER EACH WEAR)


Thoroughly clean the side of the liner without the logo. Apply a body soap by hand or with a soft cloth or sponge.

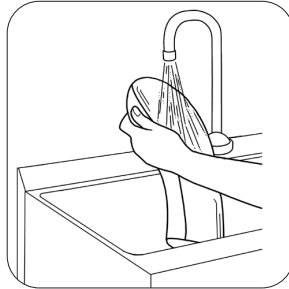Rinse all the soap from the liner with water.


Dry the liner with a clean, lint-free cloth.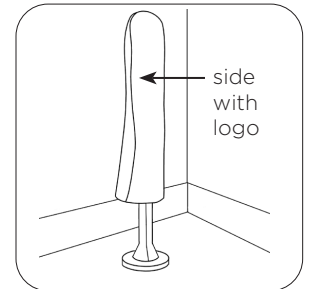

Place the liner, logo side out, onto the Drying Stand* or onto a clean, dry, flat surface.

Note: Do not scrub the Alpha Liner. Scrubbing can roughen the surface of the liner, which can then irritate the skin.

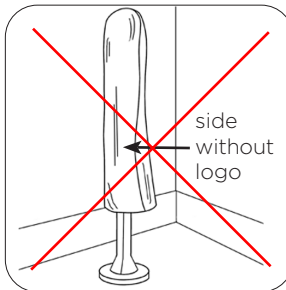
Caution: Drying the liner with the logo facing in will damage the liner.

**Available separately in most countries.*


DISINFECTING (ONCE A WEEK)


Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth. Lightly wipe the side of the liner without the logo with the cloth for two minutes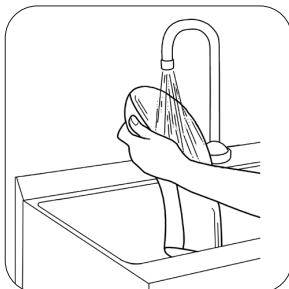

Rinse the alcohol from the liner with water.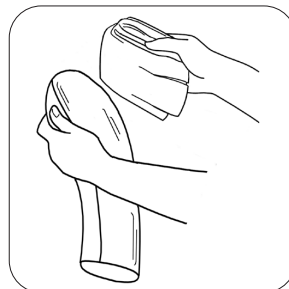

Dry the liner with a clean, lint-free cloth.

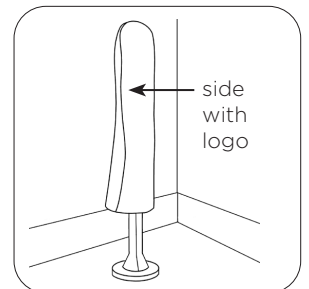Place the liner, logo side out, onto the Drying Stand* or onto a clean, dry, flat surface.

Note: Do not immerse the liner in ethyl or isopropyl alcohol. Extended contact with large amounts of ethyl or isopropyl alcohol will stiffen the liner.

SMARTTEMP LINERS

The Phase Change Materials (PCMs) in the SmartTemp Liner work by storing body heat and preventing a rise in temperature until they can no longer store any more body heat. Once they have stored all the body heat they can, they become saturated, and the temperature in the liner will begin to behave like any other prosthetic liner. Please instruct your patient that, in order to receive the benefits of the PCMs in the liner again, this stored body heat needs to be discharged from the liner as follows:

- Remove the liner from the limb.
- Place the liner in a cold or air-conditioned environment for a few hours. Simply running the liner under cold water for a brief time will not completely discharge the stored heat energy. Submerging the liner in a cold basin of water for an extended period of time would be more appropriate.

Note: the conditions required to discharge the stored body heat will vary depending on how much energy has been stored in the PCMs and the environment used to discharge the PCMs.

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WillowWood Global LLC
Ohio Willow Wood Company B.V.