

How to order a Duplicate Alpha DESIGN® Liner through OMEGA® Tracer®:

The most efficient and cost-effective way to create Alpha DESIGN Liners is still by using WillowWood's OMEGA Tracer system.

1. Start *Tracer*
2. On the main *Tracer* menu, go to **Ordering/Fabrication > View/Manage Orders**
3. Select the previously ordered liner that you wish to re-order
4. Click the **Re-order DESIGN Liner** button
5. Complete the ordering wizard (qty, notes, shipping, PO, etc)
6. Click **Send Now**

For questions or to order by phone, please contact WillowWood at 800.848.4930. When ordering, please have the serial number, file name, or purchase order number of the liner you wish to duplicate.