WILLOW WOOD®

META® Flow

Instructions for Use

META® Flow™ prosthetic foot features the META-Unibody® prosthetic foot technology and an active dorsiflexion ankle providing 14° of energy efficient ankle movement. The META Flow is designed for non-invasive single patient use, is waterproof, and is available in carbon.

WHAT'S IN THE BOX

Meta Flow Prosthetic Foot
Adjustment Hex Wrench
Adjustment Guide Card
Bench Alignment Wedge
Spectra Sock
Instructions
Patient Advisory Warning

Additional Materials Required

Meta Flow Foot Shell


INDICATIONS FOR USE

- Lower extremity prosthesis users

CONTRAINdicATIONS FOR USE

- Patients who do not have clearance for the foot.
- Patients who exceed weight rating of the foot.
- Patients who lack sufficient strength to control energy response of the ankle.

ALIGNMENT

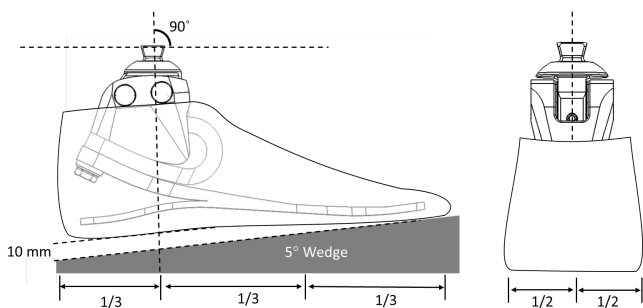STIFFNESS ADJUSTMENT

TO SPEED UP PLANTARFLEXION
[DECREASE STIFFNESS]

STEP 2

Tighten posterior set screw to decrease spring compression.

TO SLOW DOWN PLANTARFLEXION
[INCREASE STIFFNESS]

STEP 2

Tighten anterior set screw to increase spring compression.

STEP 3

Hand tighten to 3Nm
Confirm both set screws are tight.

MAINTENANCE AND CLEANING

If you expose the Foot to sand, dirt, debris, or salt water:

1. Remove the Foot Shell and Foot Sock.
2. Clean out the sand.
3. Clean the Foot, the Foot Shell, and the Foot Sock with soap and water.
4. Allow the Foot Shell and Foot Sock to dry before reassembling.

If excessive debris is noticed within the META Flow ankle, you can rinse with fresh water or compressed air to remove the debris. It may help to rotate the ankle unit while rinsing with fresh water or blowing air.

If compressed air is used, wear proper PPE.

The foot is not field serviceable and there is no recommended maintenance schedule.

WARRANTY

The warranty for the Foot is three years from date of patient fitting. Use of the Foot for amputees whose modified body weight is more than 166 kg (365 lbs) or who engage in extremely high and abusive activity is against WillowWood's recommendations and will void the three year warranty. Modified body weight is defined as the weight of the amputee plus any loads carried by the amputee. "Extremely high and abusive activities" are defined as activities that could result in injury to an individual's natural feet; and activities that expose the prosthesis to corrosives such as salt water without adequate rinsing following exposure.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

REPORTING A SERIOUS INCIDENT

In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient's, user's or other person's state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/user is established.

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