

# WillowWood SACH Foot with Titanium Pyramid Instructions

## WHAT'S IN THE BOX

WillowWood SACH Foot with Titanium Pyramid  
3 Heel Inserts (Soft, Medium, Firm)  
Instructions  
Patient Advisory Warning

## MODIFYING THE FOOT

The urethane foam shell may be lightly sanded with little resulting loss of strength. Do not expose the internal components of the foot, as this could reduce the structural integrity of the foot.

## EXPOSURE TO WATER

The WillowWood SACH Foot with Titanium Pyramid is essentially water resistant, provided no open cracks have developed and no area of the foot has been sanded. Complete submersion in water is not recommended.

## HEEL INSERTS

Install heel inserts by pressing them into the recess on the bottom surface of the foot. No adhesive is necessary for definitive use. Do not use the foot without a heel insert in place.

## PATIENT ADVISORY WARNING

The attached Patient Advisory Warning enables you, the prosthetist, to effectively notify your patients of the limitations of the components in their prosthesis, and of the need to monitor their weight and activity levels. Please review the Patient Advisory Warning with the patient upon delivery of a prosthesis with an WillowWood SACH Foot with Titanium Pyramid. The patient and the prosthetist should then sign the Patient Advisory Warning to acknowledge that it has been reviewed and understood by both parties. Give one signed copy to the patient and place one copy in the patient's file.

If a patient's weight or activity level increases after receiving a prosthesis with an WillowWood SACH Foot with Titanium Pyramid, the patient should immediately contact the prosthetist to determine whether replacement components are necessary. If a patient continues to use a prosthesis with an WillowWood SACH Foot with Titanium Pyramid after experiencing an increase in weight and/or activity level, the foot could fail with the possibility of serious injury to the patient.

To ensure that the correct components are selected for each patient, the prosthetist should weigh the patient on scales in the prosthetist's office. Do not rely on the patient's estimate of his/her own weight. Instruct the patient to monitor his/her weight weekly to ensure that it remains in a range appropriate for the prosthetic components being used.

## WARRANTY

The warranty for the WillowWood SACH Foot with Titanium Pyramid is one year from date of invoice. Use of the WillowWood SACH Foot with Titanium Pyramid for amputees whose adjusted body weight is more than 250 lbs (113 kg) or who engage in extremely high and abusive activity is against WillowWood's recommendations and will void the one year warranty. Adjusted body weight is defined as the weight of the amputee plus any loads carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural feet; and activities that expose the prosthesis to corrosives such as salt water.

## WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

## WILLOWWOOD RETENTION OF RIGHTS


WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

## WILLOWWOOD

The Ohio Willow Wood Company  
15441 Scioto Darby Road  
Mt. Sterling, OH 43143  
phone 740.869.3377 / 800.848.4930  
fax 740.869.4374 [www.willowwoodco.com](http://www.willowwoodco.com)  
Patent [www.willowwoodco.com/education-and-resources/patents](http://www.willowwoodco.com/education-and-resources/patents)



Ohio Willow Wood Company B.V.  
Keizersgracht 62/64  
1015 CS Amsterdam  
The Netherlands

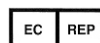

PN-1114-I 24 JAN 2014