WILLOWWOOD®

META CORE & META CORE LP

INSTRUCTIONS

Device Description

The META Core and META Core LP feet are prosthetic feet featuring the META-Unibody® platform, which provides both smooth rollover and energy return. The META Core and META Core LP feet are waterproof and designed for non-invasive, single patient use. The META Core is available in carbon or fiberglass. The META Core LP is available in carbon.

WHAT'S IN THE BOX

META Core or META Core LP Foot
Spectra Sock
Instructions
Patient Advisory Warning

Additional Materials Required

Foot Shell
Loctite 242 (or equivalent)

INDICATIONS FOR USE

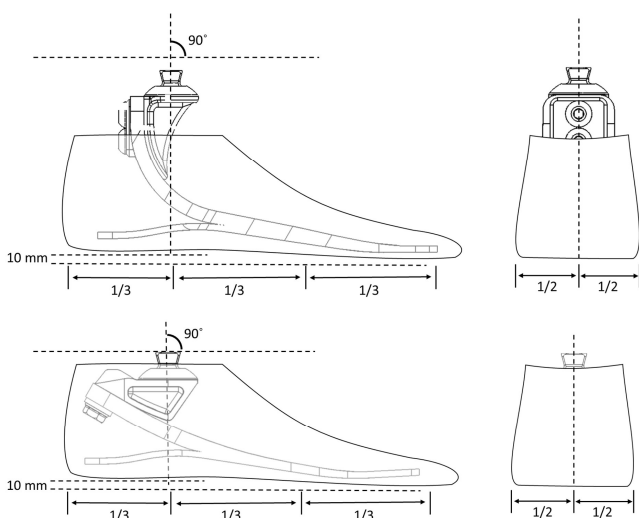
Moderate to high activity lower extremity prosthesis users

CONTRAINDICATIONS FOR USE

Patients who do not have clearance for the foot.

Patients who exceed weight rating of the foot.

ALIGNMENT

MAINTENANCE AND CLEANING

If you expose the foot to sand, dirt, debris, or salt water:

1. Remove the Foot Shell and Foot Sock.
2. Clean out the debris.
3. Clean the Foot, the Foot Shell, and the Foot Sock with body soap and water.
4. Allow the Foot Shell and Foot Sock to dry before reassembling.

The foot is not field serviceable and there is no recommended maintenance schedule.

WARRANTY

The warranty for the Foot is three years from date of patient fitting. Use of the Foot for amputees whose modified body weight is more than 166 kg (365 lbs) or who engage in extremely high and abusive activity is against WillowWood's recommendations and will void the three year warranty. Modified body weight is defined as the weight of the amputee plus any loads carried by the amputee. "Extremely high and abusive activities" are defined as activities such as skydiving, karate, and judo; activities that could result in injury to an individual's natural feet; and activities that expose the prosthesis to corrosives such as salt water without adequate rinsing following exposure.

WARRANTY DISCLAIMER

WillowWood warrants that each product manufactured will, at the time of delivery, be of workmanlike quality and substantially free of defects. WILLOWWOOD MAKES NO OTHER WARRANTY, IMPLIED, OR EXPRESSED, AND MAKES NO WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. This warranty shall terminate immediately upon an action to combine our products with other materials or in any manner to change the nature of our products. The sole remedy is replacement of the products or credit for the products. WillowWood's liability shall not exceed the purchase price of the product. WillowWood shall not be liable for any indirect, incidental, or consequential damage.

WILLOWWOOD RETENTION OF RIGHTS

WillowWood retains all intellectual property rights reflected or incorporated in its physical products, regardless of the transfer of the physical products to another party or parties.

REPORTING A SERIOUS INCIDENT

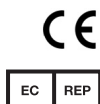
In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient's, user's or other person's state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/user is established.

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
WILLOWWOOD®

WillowWood Global LLC
15441 Sciooto Darby Road
Mt. Sterling, OH 43143
USA

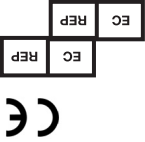telephone 740.869.3377 / 800.848.4930
fax 740.869.4374
patent www.willowwood.com/patents

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