Patient Instructions

- ALPHA Basic® Liners
- ALPHA Classic® Liners
- ALPHA Duo® Liners
- ALPHA Hybrid® Liners
- ALPHA Silicone® Liners
- ALPHA SmartTemp® Liners
- WillowWood® Express Liners
- ALPHA TruSeal® Liners

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- Liners ALPHA Basic®
- Liners ALPHA Classic®
- Liners ALPHA Duo®
- Liners ALPHA Hybrid®
- Liners ALPHA Silicone®
- Liners ALPHA SmartTemp®
- Liners WillowWood® Express
- Liners ALPHA TruSeal®

Patient Instructions

- ALPHA Basic® Liners
- ALPHA Classic® Liners
- ALPHA Duo® Liners
- ALPHA Hybrid® Liners
- ALPHA Silicone® Liners
- ALPHA SmartTemp® Liners
- WillowWood® Express Liners
- ALPHA TruSeal® Liners

Thank you for using this liner. In order to ensure the highest level of comfort and longest possible life from your liner, please take the following steps:

- Read and follow these instructions.
- Inspect your liners after each day of use. Watch for signs of unusual wear. This may indicate changes in the prosthetic fit. If you see unusual wear, call your prosthetist.
- Call your prosthetist if you have any questions about your liners.

WILLOWWOOD®

WillowWood Global LLC
15441 Scioto Darby Road
Mt. Sterling, OH 43143
téléphone 740.869.3377 / 800.848.4930
télécopieur 740.869.4374 www.willowwood.com
Brevet www.willowwood.com/patent

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Electronic Instructions for Use are available at www.willowwood.com
A paper copy of the Instructions for Use is available upon request at no additional cost.

PUTTING ON YOUR LINER

Before putting on your liner, make sure that your limb is clean, dry, and free of soap residue. Cover open wounds and non-intact skin with a gauze patch. **Do not apply any type of lotion or powder to the residual limb or to the liner.** These products could damage the liner.


Invert the liner so that the logo is on the inside,. Place the end of the liner against the end of your limb. If using a Locking Liner, center the pin attachment on the bottom of your limb.

Position the liner as follows:


Liner Type	Logo Position
Classic AK Liner AK DESIGN® Liner	Logo on lateral side (outside).
Duo Liner Basic Liner Hybrid AK Liner Silicone AK Liner SmartTemp AK Liner Express AK Liner	The position of the logo does not matter. Varying the orientation from day to day can prolong the life of the liner.
All other liners	Logo in front.

Carefully roll the liner up onto your limb.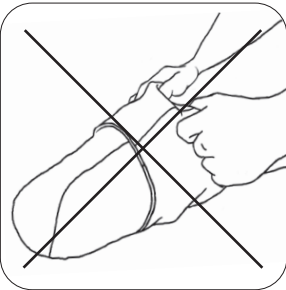


DO NOT PULL the liner onto the limb. Pulling the liner may stretch the skin and result in an uncomfortable fit.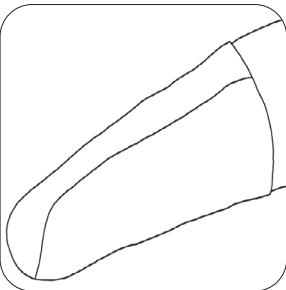

Make sure that there are no wrinkles or air pockets between the liner and the limb.


DAILY CLEANING

Good hygiene is very important when using the liner. The liner must be thoroughly cleaned at the end of each day of wear and disinfected once each week. Having two liners is recommended so that you can wear one while cleaning and drying the other.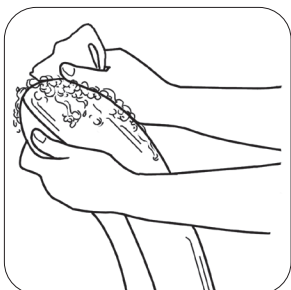

Thoroughly clean the side of the liner without the logo. Use lukewarm tap water and a body soap that does not irritate your skin. Apply the soap with a clean, soft cloth or sponge.


Note: Do not scrub the liner. Scrubbing can roughen the surface of the liner, which can then irritate the skin.

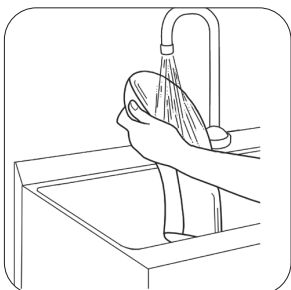Rinse all the soap residue from the liner with water.


Dry the liner with a clean, lint-free cloth.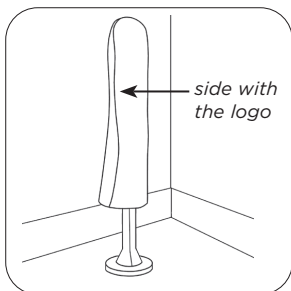

Place the liner, **logo side out**, onto the Drying Stand* or onto a clean, dry, flat surface.

Clean your limb with soap and rinse well.

**Available separately in most countries*

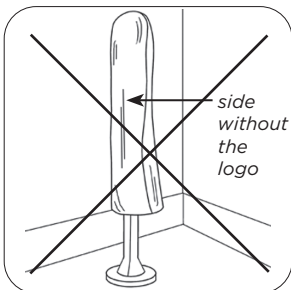
Caution: Drying the liner with the logo facing in will damage the liner.

WEEKLY DISINFECTING


Place a small amount of ethyl or isopropyl alcohol on a soft, clean cloth. Lightly wipe the side of the liner without the logo with the cloth for two minutes.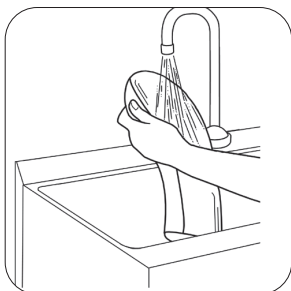

Rinse the alcohol from the liner with water.

Dry the liner with a clean, lint-free cloth.

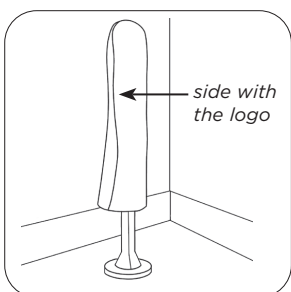Place the liner, **logo side out**, onto the Drying Stand* or onto a clean, dry, flat surface.

Note: Do not soak the liner in the ethyl or isopropyl alcohol. Soaking the liner in alcohol will stiffen the liner.

**Available separately in most countries*

STORING THE LINER

When the liner is not being worn, it should be stored logo side out in a cool, dry place. If the liner will not be used for a long time, cover the dry liner with a clean plastic bag.

PERSPIRATION

Some activities could cause you to sweat heavily. If this happens, occasionally dry off the liner and the limb. This helps to keep the liner from slipping on the limb.

WHAT YOU SHOULD KNOW ABOUT MANAGING VOLUME LOSS

It is completely normal for the volume of your limb to change during the day. When your limb shrinks, the socket no longer fits as snugly as it should. As a result, your socket may slip down your leg, and your limb may move around inside of your socket. This could cause your liner to wear out prematurely.

If the volume of your limb changes, ask your prosthetist about the best solution for you.

ADDITIONAL INFORMATION FOR SPECIFIC LINER TYPES

ALPHA DUO LINER

If you have an Alpha Duo Liner, you will also need to put on a One Gel Sock.

After you have put on the liner, invert the One Gel Sock so that fabric side is facing in. Place the end of the inverted sock against the end of the liner.

Roll the sock up onto the limb so that the fabric side is facing out. Make sure that there are no wrinkles or air pockets between the One Gel Sock and the liner.

ALPHA SMARTTEMP LINER

Your liner stores heat from your body or the air to help keep the liner temperature from rising. There is a limit to how much heat the liner can store. Once the liner has stored all the heat it can, you should enter a cooler environment. If the liner remains saturated with heat, you may discharge the heat as follows:

- Remove the liner from your limb.
- Place the liner in a cold or air-conditioned environment for a few hours. You may speed up the release of heat from the liner by putting the liner in a cold basin of water.

Note: The time it takes to release the heat depends on how much heat was stored and how you are releasing it.

POSSIBLE LINER ISSUES

Problem	Probable Cause
Locking Liner is “pistoning” up and down in your socket.	Your limb may have shrunk.
Top edge of liner is rolling down your leg.	Liner is possibly too short or too tight.
Heavy sweating (if you have just recently started wearing the liner).	Your body has not yet adjusted to the liner.
Heavy sweating after wearing the liner a long time without sweating.	Air is getting between the liner and the residual limb. This is possibly because your limb has shrunk.
	There is a hole in the liner.
Heavy sweating (if you are wearing a SmartTemp Liner).	The SmartTemp Liner cannot store any more body heat.
Skin irritation all over your limb.	Cleaning or disinfecting your liner the wrong way.
	Using a harsh or irritating lotion, cream, powder, or soap.
	Washing the limb too frequently.
	Infection of the limb and liner.
	The liner or the socket is loose, possibly because your limb has shrunk.
Skin irritation along the top edge of your liner.	The liner is being pulled onto the limb instead of being rolled onto the limb, or the liner is too tight.
Open wounds and non-intact skin.	Call your prosthetist right away.

Solution

Call your prosthetist

Call your prosthetist.

If you are still sweating after several weeks of liner use, call your prosthetist.

Call your prosthetist.

Call your prosthetist.

Discharge the body heat in a cool environment.

Make sure that you are following the proper cleaning, rinsing, and disinfection procedures outlined in this pamphlet.

Have you recently started using a different lotion, cream, powder, or soap? If so, stop using that product.

Washing the limb more than once per day can result in increased levels of skin irritation.

Call your prosthetist. You may also need to call a dermatologist.

Call your prosthetist.

Refer to the instructions in this pamphlet for putting on your liner. If the problem continues, call your prosthetist.

Follow the instructions your prosthetist gave you. Never use a liner in direct contact with open sores or non-intact skin. If you must wear the liner over open sores or non-intact skin, wear a bandage over the wound.

If you do not understand these warnings, please discuss them with your prosthetist.

Caution: Alpha Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.

Caution: This Alpha product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.

Warning: To avoid danger of suffocation, keep this product away from babies or children.

Failure to follow WillowWood instructions will void the warranty.

Since WillowWood believes that the liners must be fit and dispensed by a qualified professional, please understand that WillowWood will not ship liners directly to an amputee's home.

Warning for Locking Liner Users:

Your prosthetist should securely attach a pin to each of your Locking Liners. You must check the pin each time you put on your prosthesis to make sure that the pin is still secure. If you do not do this, the pin could come out of the liner, and you might fall and injure yourself. Use of a sleeve (or similar) is also recommended.

REPORTING A SERIOUS INCIDENT

In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient's, user's or other person's state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/user is established.

Device Description

Designed for comfort and performance, the Alpha Truseal Liner is made from proprietary, platinum cured grade silicone. The four integrated fins are designed to ensure confident suction suspension while the satin finish allows easy donning and doffing

WHAT'S IN THE BOX

ALPHA Truseal	S494-11XX
	S495-11XX

**Value of X depends on size ordered.*

WILLOWWOOD®

WillowWood Global LLC
15441 Scioto Darby Road
Mt. Sterling, OH 43143
téléphone 740.869.3377 / 800.848.4930
télécopieur 740.869.4374 www.willowwood.com
Brevet www.willowwood.com/patent

Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands

Electronic Instructions for Use are available at www.willowwood.com.
A paper copy of the Instructions for Use is available upon request at no additional cost.

DONNING

- Invert the liner such that the fins are on the inside
- Place the liner against the bottom of the limb
- Roll the liner onto the limb taking care to avoid trapping air or creating wrinkles
- Step into your socket and allow air to be expelled from the expulsion valve

MAINTENANCE AND CLEANING

- Inspect daily for holes, tears, and premature wear
- Keep the surfaces clean and clear of debris using mild soap and dry with a lint free cloth
- Disinfect the surfaces weekly using isopropyl alcohol and a lint free cloth

WARRANTY

WillowWood Liners are warrantied for 12 months, against defects in material and workmanship, from the date of patient fitting.

REPORTING A SERIOUS INCIDENT

In the event of a serious incident occurring as a direct consequence of using this medical device, which either directly, or indirectly caused (a) the death of the patient/user or another person, (b) the temporary or permanent serious deterioration of the patient's, user's or other person's state of health or (c) a serious public health threat, please immediately report the incident by contacting WillowWood Global or, if in the EU, the authorized European Union representative and the competent authority within the member state in which the patient/user is established.

ALPHA Products can melt or burn if exposed to high temperatures or flame. Do not expose your Alpha product to these conditions.

This ALPHA Product is intended for use on a single patient. Use of the product with multiple patients could lead to cross contamination between patients.

WARNING: To avoid danger of suffocation, keep this product away from babies or children.

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- Liners ALPHA Basic®
- Liners ALPHA Classic®
- Liners ALPHA Duo®
- Liners ALPHA Hybrid®
- Liners ALPHA Silicone®
- ALPHA SmartTemp® Liners
- Liners WillowWood® Express
- Liners ALPHA TruSeal®

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